



Single Patient Use Slide Tubes


How To Use

- SlideMates should be removed after transfers are completed
- SlideMates are single patient use, but can remain with the patient throughout their entire hospital stay

**Always Follow your Safe Patient Handling Protocols
when using Slide Sheets**



1. Log roll the patient and position the SlideMate under the patient and draw sheet or bed sheet.

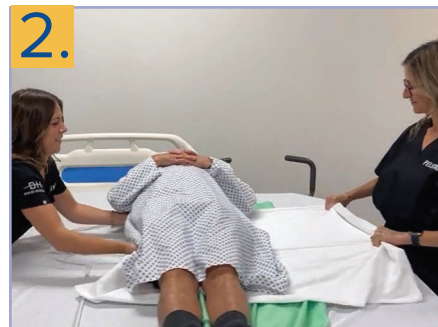

2. Caregiver 1 pushes the patient while Caregiver 2 pulls the draw sheet or bed sheet to initiate the transfer.



3. Use the draw or bed sheet to reposition the patient to the center of the bed or stretcher.



4. Remove the SlideMate by gently pulling it out from beneath the patient, working either from head to toe or toe to head. Always remove the SlideMate immediately after the transfer and repositioning is complete.

Need a Quick
SlideMate Demo?



www.bridgehcusa.com