

# BRIDGEAIR™

## Single Patient Use Lateral Transfer Mattress

### How To Use

Need a Quick  
BridgeAIR Demo?



Always Follow your Safe Patient Handling Protocols  
when using BridgeAIR™




1. Log roll the patient on their side, towards Caregiver 1, and unfold and position BridgeAIR™ underneath the patient and sheet, making sure the patient is centered on the center line.



2. Log roll the patient to their other side, towards Caregiver 2 and finish unrolling the mattress.



3. Secure the patient by loosely connecting the safety straps.

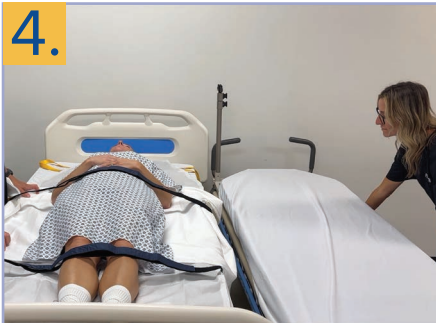


4. Bring the receiving surface next to the bed and ensure transfer surfaces are close together (within 3").



5. Lock the wheels on both the bed and the receiving surface.



6. Connect and secure the air hose to the air inlet port on the side of the mattress where the transfer is taking place.

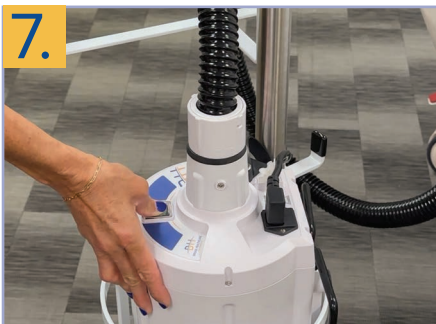

7. Inform the patient that the mattress will be inflating and press the On/Off button to turn on the air pump.  
*Note: Never leave the patient unattended while the mattress is inflated*




8. Use the transfer handles to push/pull the patient from one surface to the other. Check that the patient is centered on the receiving surface before turning off the pump.



9. If repositioning, gently pull on the handles as needed.



10. Turn off the pump to deflate the mattress. Disconnect the safety straps and place them under the bed/stretcher mattress.

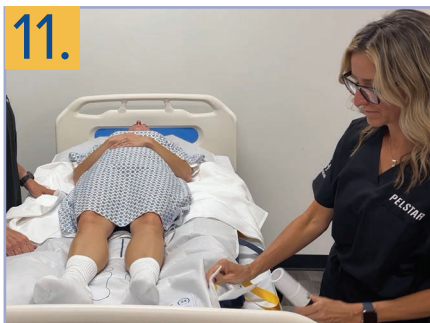

11. Disconnect the air pump from the mattress.



12. If using bed connecting straps, securely attach them to the bed.