

# BRIDGE STAR SYSTEMS

## *SAFE TURNING AND REPOSITIONING*

### Instructions for Use

#### Repositioning Sheet

- 1 Place unfolded sheet next to patient/resident with tag side up and tag towards the foot end.
- 2 If using with BreatheDRY absorbent pad, place pad on top of the sheet.
- 3 Ensure the top of the sheet is aligned with top of patient/resident shoulders.
- 4 Log roll patient/resident away from sheet.
- 5 Tuck sheet underneath patient/resident so they will be centered on sheet when returned to supine position.
- 6 Log roll patient/resident the opposite way to pull through and flatten sheet ensuring no folds or creases remain underneath patient/resident.
- 7 Return patient/resident to supine position and use sheet handles to reposition patient/resident on the bed.

#### Cleaning and Reprocessing

This product is designated Single Patient Use, meaning it is intended to remain with the same patient throughout their stay. It can be discarded after use or cleaned and reprocessed in accordance with facility infection control protocols.

For cleaning instructions go to:  
[www.bridgehcusa.com/resources/user-manuals/](http://www.bridgehcusa.com/resources/user-manuals/)

**Always Follow your Safe Patient Handling Protocols  
when using Repositioning Sheets and Wedges**



**PELSTAR LLC**

**T** (800) 815-6615  
**E** info@pelstarllc.com  
**W** bridgehcusa.com

9500 West 55th Street McCook, IL 60525 USA

# BRIDGE STAR SYSTEMS

## *SAFE TURNING AND REPOSITIONING*

### Instructions for Use

#### Turning and Repositioning Wedges

- 1 Log roll patient to off-load pressure.
- 2 Place two wedges underneath sheet; arrow side down, a hands width apart, one above sacral area, one below sacral area.
- 3 Gently log roll patient onto wedges to achieve sacral off loading.
- 4 Ensure sheet is smooth with no wrinkles underneath patient/resident.

#### Cleaning and Reprocessing

This product is designated Single Patient Use, meaning it is intended to remain with the same patient throughout their stay. It can be discarded after use or cleaned and reprocessed in accordance with facility infection control protocols.

Made from clinical-grade materials, it can be wiped down using hospital-approved disinfectants, including a 1:10 diluted bleach solution.

This product is not machine washable.

**Always Follow your Safe Patient Handling Protocols  
when using Wedges**



**PELSTAR LLC**

**T** (800) 815-6615  
**E** info@pelstarllc.com  
**W** bridgehcusa.com

9500 West 55th Street McCook, IL 60525 USA